

Transseptal Transcatheter Mitral Valve Replacement in Patients with Mitral Annular Calcification: ENCIRCLE MAC Registry One-year Outcomes


Prospective, multicenter, single-arm study

Objectives

To evaluate the safety and effectiveness of transseptal transcatheter mitral valve replacement (TMVR) with the SAPIEN M3 system in patients with symptomatic MAC-related MVD, including moderate-to-severe or greater MR, severe MS or combined moderate MR with at least moderate MS, who were deemed unsuitable for surgery or transcatheter repair by the local heart team.

Methods

- 36 enrolling sites in the United States, Canada, the Netherlands, the United Kingdom, and Australia¹
- Primary endpoint was a non-hierarchical composite of all-cause mortality or HF hospitalization at 1 year*
- Secondary endpoints included the following at 1 year compared to baseline*¹:
 - Improvement in MR severity
 - Improvement in NYHA functional class
 - Improvement in KCCQ-OS score
 - Improvement in LVEDVi

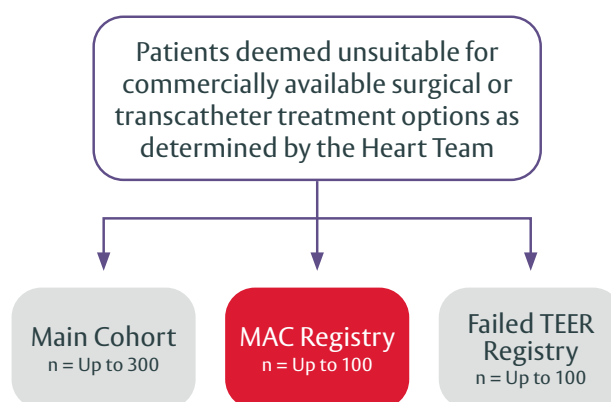

Key Outcomes


- Significant and sustained MR reduction with **98.5%** of patients achieving MR ≤1+ at 1 year
- Significant functional improvements with **87.7%** of patients achieving NYHA class I/II at 1 year
- Significant quality of life improvements with an **18-point** increase in KCCQ-OS score at 1 year

*The sample size for these registries were not statistically derived. No hypotheses are defined for these cohorts.

Baseline characteristics


Demographics and Medical History	N=100
Mean age (years)	76.1
Male	51.0%
STS score, mitral valve replacement	8.4%
MV mean gradient (mmHg)	6.1
LVEF	54.6%
Congestive heart failure	68.0%
Prior CABG	28.0%
Prior stroke	14.0%
Prior mitral valve intervention	3.0%
PPM or ICD	38.0%
Hypertension	91.0%
Pulmonary hypertension	44.0%
Diabetes	39.0%
Atrial fibrillation	66.0%
NYHA class III/IV	73.0%
Mitral regurgitation and mitral stenosis summary*	
MR ≥3+	71.0%
MR =2+ and ≥moderate stenosis	19.0%
Severe stenosis	10.0%

Results

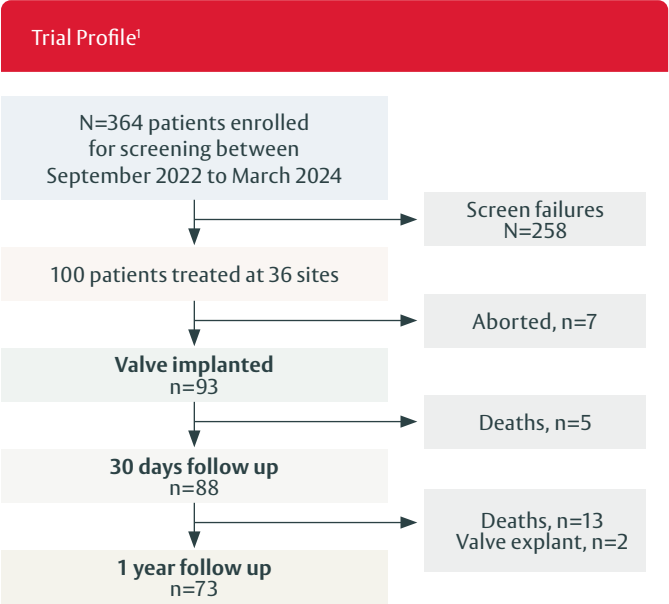

*The worst case among Transesophageal Echo (TEE) and Transthoracic Echo (TTE) was used as the baseline; TTE was used for all other visits.
†Data are presented as % or Median (Q1, Q3).
‡Defined as the time from femoral vein access to guide sheath removal.
§Defined as the time from guide sheath insertion to removal.
**Defined as the time from dock delivery system insertion to removal.
††Defined as the time from Commander delivery system insertion to removal.
‡‡Routine closures include all that were not clinically significant or were performed under standard clinical practice (i.e., preventative or planned based on patients' history).
§§Defined as ≥ 30 mmHg increase in peak LVOT gradient attributable to TMVR resulting in peak LVOT gradient ≥ 50 mmHg and hemodynamic compromise or symptoms attributable to LVOT obstruction.

Multifaceted and advanced MVD: MAC with MR and/or MS

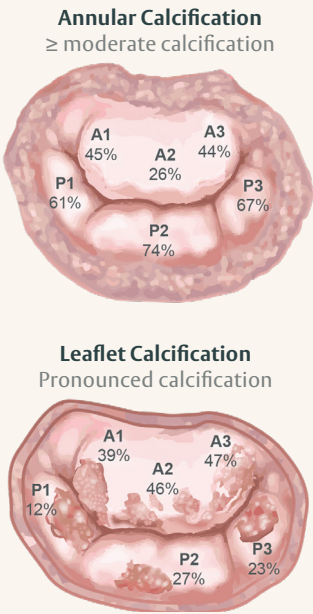


Includes patients with calcium thickness ≥10 mm, calcium distribution ≥ 270°, calcium on both trigones and both leaflets

Procedural Information ^{†1}	SAPIEN M3 N=100
Procedure time [†] (min)	119.5 (97.0, 149.0)
Device time [†]	97.0 (78.0, 119.0)
Dock deployment time ^{**}	61.0 (45.0, 86.0)
Valve implant time ^{††}	9.0 (7.0, 11.0)
Procedure aborted	7.0%
Conversion to surgery	1.0%
Valve embolization	1.0%
PVL closure	12.0%
ASD closure ^{††}	
Routine closure	12.0%
Clinically significant closure	13.0%
Intraprocedural stroke	0.0%
Clinically significant LVOT obstruction ^{§§}	1.0%

Primary composite endpoint of all-cause mortality and heart failure hospitalization¹

Figure 1: All-Cause Mortality or HFH

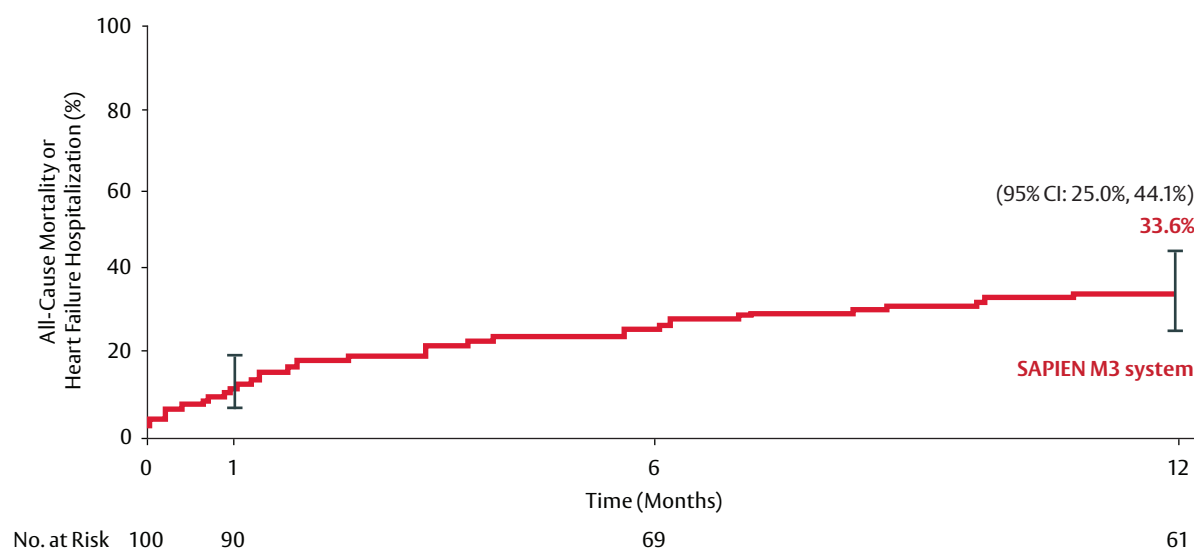

Proven safety profile in highly complex and comorbid MAC patients¹

Safety Outcomes*	30 Days	1 Year
All-cause mortality	6.0%	21.1%
HF hospitalization	5.2%	19.4%
Stroke	1.0%	7.4%
• Disabling stroke	0.0%	5.1%
• Non-disabling stroke	1.0%	2.3%
New atrial fibrillation	12.1%	21.7%
New PPM	6.6%	10.9%
Major bleeding or above, MVARC [†]	11.0%	21.1%
Mitral valve reintervention [‡]	3.1%	11.4%
Clinically significant valve thrombosis [§]	1.0%	2.3%
Hemolysis requiring intervention**	9.2%	9.2%

*Data are Kaplan-Meier estimates %.

[†]Major bleeding or above includes bleeding with MVARC primary bleeding scale of major, extensive, life-threatening or fatal.


[‡]Only interventions after procedure are reported: includes PVL closure (8) and surgical mitral valve replacement (2).

[§]Clinically significant valve thrombosis includes leaflet thickening with impaired leaflet motion with mitral valve stenosis (increase in mean mitral valve gradient ≥ 5 mm Hg) and clinical signs or symptoms of mitral valve stenosis.

**Hemolysis requiring blood transfusion or mitral reintervention.


Significant and sustained MR reduction through 1 year¹

Figure 2: MR Severity

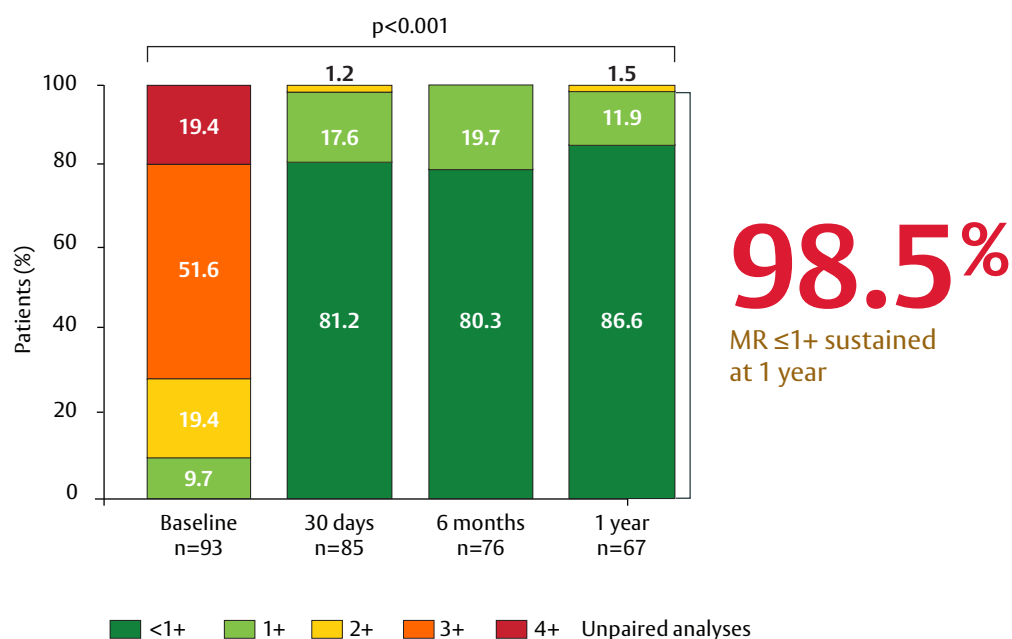

Decreased gradients in patients with moderate and severe MS sustained at 1 year¹


Figure 3: Gradients*

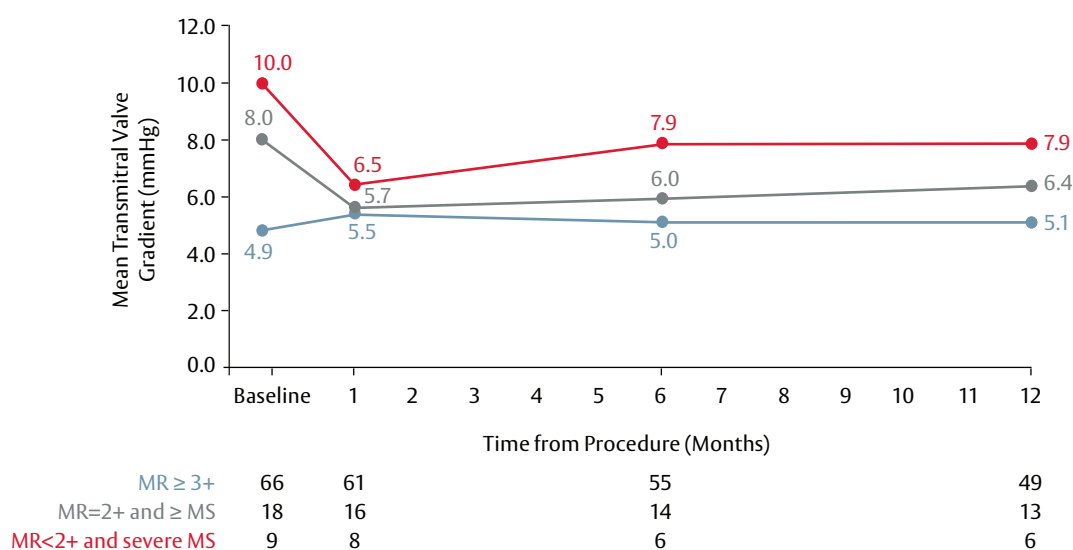

*Not a secondary endpoint.


Significant improvements in functional status¹

Figure 4: NYHA Functional Class

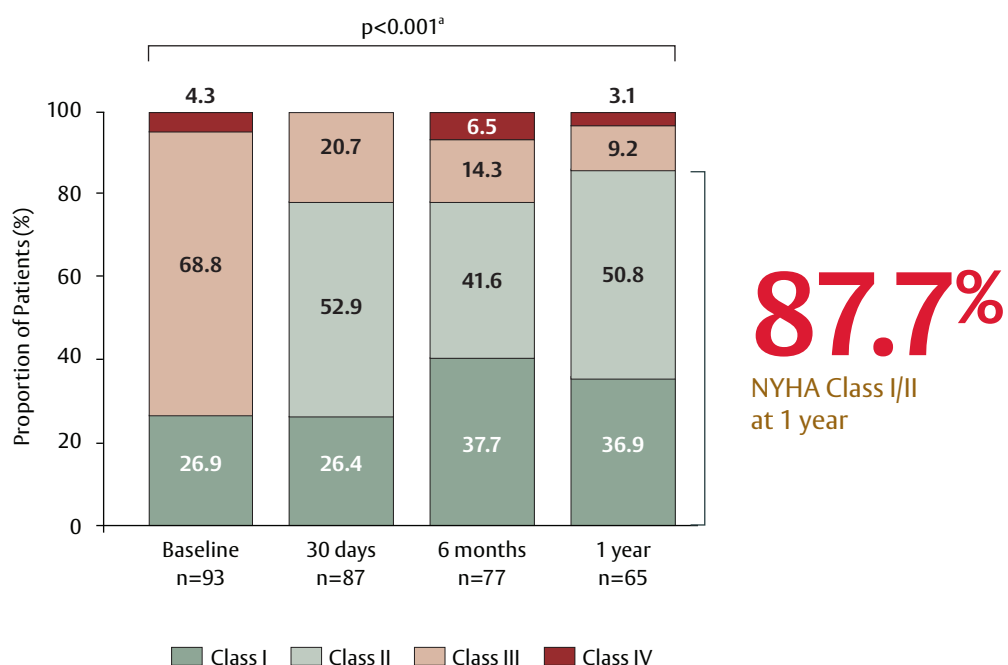

Significant quality of life improvements¹

Figure 5: KCCQ Overall Summary Score

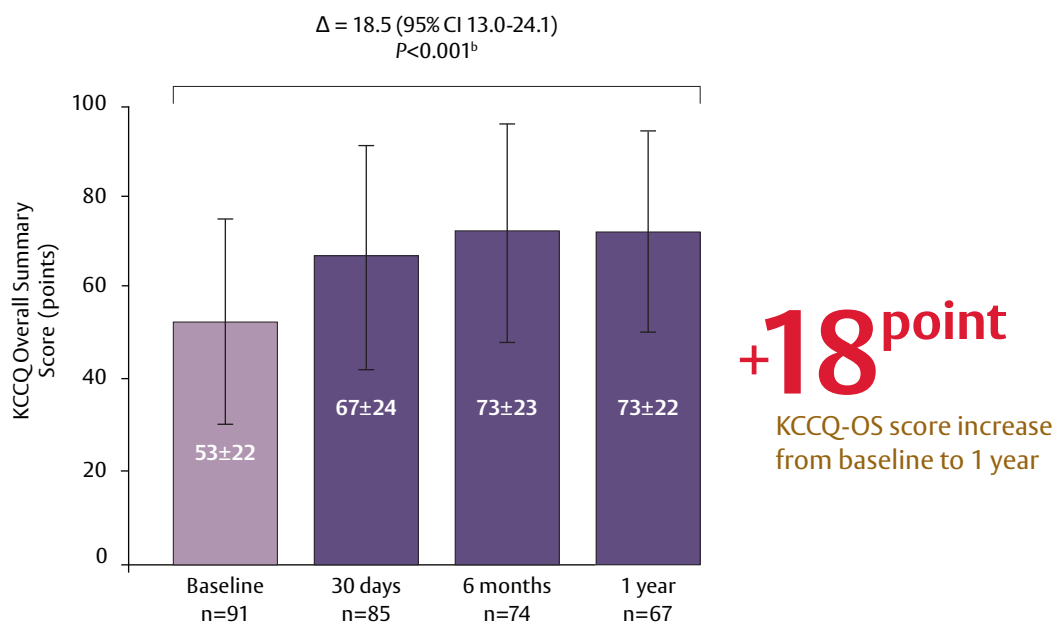

Graph shows unpaired analysis. Error bars represent 95% CI. P values calculated from paired analysis using Wilcoxon signed rank test a) baseline vs. 1 year (n=65; NYHA class I/II=87.7%) or Student's t-test, b) baseline vs. 1 year (n=67; mean baseline KCCQ=54.3; mean 1-year KCCQ=72.9)

SAPIEN M3 Transcatheter Mitral Valve Replacement System- Important Safety Information

Indications: The SAPIEN M3 transcatheter mitral valve replacement system (SAPIEN M3 system) is indicated for the treatment of symptomatic moderate-to-severe or severe mitral regurgitation (MR) in patients who are deemed unsuitable for surgery or transcatheter edge-to-edge repair (TEER) therapy by a multidisciplinary heart team. The SAPIEN M3 system is also indicated for the treatment of symptomatic mitral valve dysfunction (moderate-to-severe or severe MR, severe mitral stenosis (MS), or moderate MR with moderate MS) associated with mitral annular calcification (MAC) in patients who are deemed unsuitable for surgery or TEER therapy by a multidisciplinary heart team. The Edwards 23F guide sheath is indicated to provide venous vascular access to cardiac structures enabling the introduction and removal of SAPIEN M3 transcatheter mitral valve replacement devices.

Contraindications: The SAPIEN M3 system is contraindicated in patients who cannot tolerate any anticoagulation/antiplatelet regime or intraprocedural heparin; or who have active bacterial endocarditis or other active infections.

Warnings: The SAPIEN M3 system devices and Edwards 23F guide sheath are designed, intended, and distributed STERILE for single use only. Do not resterilize or reuse the devices. There are no data to support the sterility, non-pyrogenicity, and functionality of the devices after reprocessing. Do not mishandle the SAPIEN M3 system devices or use them if the packaging or any components are not sterile, have been opened or are damaged (e.g., kinked or stretched), or the expiration date has elapsed. Patients with hypersensitivities to cobalt, nitinol (nickel or titanium), chromium, molybdenum, manganese, silicon, bovine tissue, and/or polymeric materials may have an allergic reaction/immunological response to these materials. Accelerated deterioration of the valve may occur in patients with altered calcium metabolism. Exercise caution when implanting a valve in patients with clinically significant coronary artery disease as it may result in myocardial ischemia. Prior to delivery, the valve must always remain hydrated and cannot be exposed to solutions other than its shipping storage solution and sterile physiologic rinsing solution. Valve leaflets mishandled or damaged during any part of the procedure will require replacement of the valve. Do not use the valve if the tamper-evident seal is broken, the storage solution does not completely cover the valve, the temperature indicator has been activated, the valve is damaged, or the expiration date has elapsed. Do not add or apply antibiotics to the storage solution, rinse solutions, or the valve. The physician must verify correct orientation of the valve prior to its implantation. The procedure should be conducted under 3D echocardiography and fluoroscopic guidance. Some fluoroscopically guided procedures are associated with a risk of radiation injury to the skin. These injuries may be painful, disfiguring, and long-lasting. Use of excessive contrast media may lead to renal failure. Measure the patient's creatinine level prior to the procedure. Contrast media usage should be monitored. Observation of the pacing lead throughout the procedure is essential to avoid the potential risk of pacing lead perforation. In the event of device malfunction or device damage during use (e.g., destructive deformation to the catheter, balloon burst, etc.) safely remove the device(s). If unable to safely remove the device(s), conversion to surgery is recommended. Prior to valve deployment, 3D echocardiographic and fluoroscopic (short-axis view) verification must be used to confirm that the guidewire passes through the center of the implanted dock and has unrestricted movement. Failure to do so can result in chordal rupture and/or the valve being deployed outside of target location. Incorrect positioning of the dock and/or valve may lead to left ventricular outflow tract obstruction, paravalvular leak (PVL), valve migration, or valve embolization. Valve recipients must be on appropriate anticoagulation regimen, determined at the physician's discretion based on individual subject needs for a minimum of 6 months. Failure to anticoagulate and bridge appropriately will lead to valve thrombosis. For subjects receiving vitamin K antagonists, target range for INR is 2.5 to 3.5. After 6 months, continued antithrombotic therapy is recommended as tolerated. Characteristics of the device(s) to be inserted into the guide sheath should be evaluated to prevent damage to the interior liner of the guide sheath, damage to the device(s) being inserted, and/or injury to the patient. Patient injury could occur if the guide sheath is not unflexed prior to removal. In the event of device malfunction or device damage during use (e.g. destructive deformation to the catheter) safely remove the device(s). If unable to safely remove the device(s), conversion to surgery is recommended.

Precautions: Glutaraldehyde may cause irritation of the skin, eyes, nose, and throat. Avoid prolonged or repeated exposure to, or breathing of, the solution. Use only with adequate ventilation. If skin contact occurs, immediately flush the affected area with water; in the event of contact with eyes, seek immediate medical attention. For more information about glutaraldehyde exposure, refer to the Material Safety Data Sheet available from Edwards Lifesciences. Additional precautions for transseptal replacement of a mitral valve include abnormalities in the caval vein precluding safe transvenous femoral access for transseptal approach, presence of atrial septal occluder device, or calcium preventing safe transseptal access. Use caution in tortuous or calcified vessels that would prevent safe entry of the guide sheath and introducer. Patients with a pre-existing prosthesis should be evaluated for the location, shape, construction, and characteristics of the prosthesis (e.g., low-deployed aortic prosthesis, rigid or small annuloplasty ring, septal occluder, etc.) as it may interfere with SAPIEN M3 system deployment, functionality, or dock/valve durability. Patients with mitral annular calcification should be evaluated for the characteristics of the calcium and mitral pathology as it may interfere with the dock trajectory during deployment, result in malposition of the dock/valve, and/or have an increased risk of PVL. Patient's sub-valvular anatomy should be evaluated for the characteristics of papillary muscles, chordae, and ventricular wall as it may interfere with or prevent dock deployment. Patients with the following characteristics have an increased risk of PVL which may lead to hemolysis and/or intervention: compromised leaflet integrity (e.g., perforation, endocarditis, Barlow's syndrome, etc.); fail or prolapse located at the commissures; fail or prolapse located at P3 leaflet in conjunction with a commissural distance ≥ 42 mm; Any large non-commissural fail or prolapse. The sheath and introducer are coated with a hydrophilic lubricious coating. Failure to activate the hydrophilic coating with heparinized saline may result in difficulty with insertion. To maintain proper valve leaflet coaptation, do not overinflate the deployment balloon. Appropriate antibiotic prophylaxis is recommended post-procedure in patients at risk for prosthetic valve infection and endocarditis. Long-term durability has not been established for the valve. Regular medical follow-up is advised to evaluate valve performance. The safety and effectiveness of the SAPIEN M3 system have not been established for patients who have/are: a left ventricular end-diastolic diameter ≥ 75 mm; a commissural distance ≥ 50 mm; a left ventricular ejection fraction below 25%; severe RV dysfunction; History of heart transplant; Severe pulmonary hypertension; Blood dyscrasias defined as: leukopenia (WBC < 3000 cells/ μ L), acute anemia (Hb < 9 g/dL), thrombocytopenia (platelet count < 50,000 cells/ μ L), or history of bleeding diathesis or coagulopathy.

Potential Adverse Events: Potential risks associated with the anesthesia, interventional procedure, and imaging include but are not limited to: death; stroke or other neurological dysfunction; cardiovascular injury such as cardiac structure complications, vascular complications, and access related complications; heart failure or low cardiac output / worsening of heart failure; renal insufficiency or renal failure; cardiogenic shock; cardiac arrest; pericardial effusion or cardiac tamponade; thromboembolism including air, calcific valve material, or thrombus; retroperitoneal bleed; arrhythmia; hypertension or hypotension; new or worsening valvular regurgitation; bleeding / hematoma / hemorrhage; hemolysis that may require transfusion or intervention; device/valve thrombosis; respiratory insufficiency or respiratory failure; paravalvular or transvalvular leak; device deterioration (wear, fracture, calcification, or other) reoperation / reintervention; device explants; pleural effusion; LVOT obstruction; emergency cardiac surgery; conversion to cardiac surgery; thoracic bleeding; valve stenosis; myocardial infarction; pulmonary edema; transient ischemic attack including clusters; device migration, malposition or embolization; infection including septicemia and endocarditis; allergic reaction to anesthesia, contrast media, or device material; deterioration of native valve (leaflet tear/tearing, leaflet retraction, leaflet thickening, or other); structural valve deterioration (wear, fracture, calcification, leaflet tear/tearing from the stent posts, leaflet retraction, suture line disruption of components of a prosthetic valve, thickening, stenosis); nonstructural valve dysfunction; atrial septal defect; syncope; dock wear or fracture; conduction system defect which may require a permanent pacemaker; skin burn; mechanical failure of delivery system, and/or accessories; valve deployment in an unintended location; abnormal lab values (including electrolyte imbalance); angina; anemia; stroke/TIA or nerve injury; vessel spasm; and catheter entrapment; fever; inflammation; pain or changes at the access site.

CAUTION: US law restricts this device to sale by or on the order of a physician. See instructions for use for full prescribing information.

ASD, Atrial septal defect; CABG, Coronary artery bypass graft; HF, Heart failure; HFH, Heart failure hospitalization; ICD, Implantable cardioverter-defibrillator; KCCQ-OS, Kansas City Cardiomyopathy Questionnaire – Overall Summary; LV, Left ventricle; LVEDVi, Left ventricular end-diastolic volume index; LVEF, Left ventricular ejection fraction; MAC, Mitral annular calcification; MR, Mitral regurgitation; MS, Mitral stenosis; MV, Mitral valve; MVD, Mitral valve dysfunction; MVARC, Mitral Valve Academic Research Consortium; NYHA, New York Heart Association; PPM, Permanent pacemaker; PVL, Paravalvular leak; STS, Society of Thoracic Surgeons; TEER, Transcatheter edge-to-edge repair; TTE, Transthoracic echocardiogram

References:

1. Guerrero M, et al. Transseptal Transcatheter Mitral Valve Replacement in Patients with Mitral Annular Calcification: ENCIRCLE MAC Registry One-year Outcomes. Presented at New York Valves 2026.

Edwards, Edwards Lifesciences, the stylized E logo, ENCIRCLE, the ENCIRCLE logo, SAPIEN, and SAPIEN M3 are trademarks or service marks of Edwards Lifesciences Corporation or its affiliates. All other trademarks are the property of their respective owners.

© 2026 Edwards Lifesciences Corporation. All rights reserved. PP-US-12709 v1.0

Edwards Lifesciences • One Edwards Way, Irvine CA 92614 USA • edwards.com

