

# Technical Bulletin



Designed in collaboration with and validated by clinicians, the EV1000 clinical platform offers you scalability and adaptability for both the OR and ICU.

The EV1000 clinical platform enables you to choose the parameters needed to monitor your patients and may be used with a variety of Edwards Lifesciences advanced hemodynamic monitoring solutions.

## EV1000 ClearSight Clinical Platform Technical Bulletin – 002

This technical bulletin summarizes software compatibility for the EV1000 clinical platform (model number: EVPMP and PC2K), outlined in the operator's manual.

At Edwards Lifesciences, we are committed to helping advance the care and treatment of surgical and critical care patients. As a part of this pursuit, we have developed a software update to ensure the best patient monitoring experience.

### Software Compatibility:

The PC2K is the pressure controller used with the ClearSight system on the EV1000 and HemoSphere monitoring systems. While all PC2Ks look alike, PC2K included with the HemoSphere systems run software version 10.0. This software version is compatible with all HemoSphere and EV1000 monitoring systems that run software version 1.80. PC2Ks with the 10.0 software system are not compatible with EV1000s running on software version 1.70.

If the software between the PC2K and the HemoSphere or EV1000 monitoring system are incompatible, you will encounter a message on the screen indicating incompatibility. You can revert to using the pressure controller that was provided with your EV1000 clinical platform, or you can contact us and we will work with you to request a new compatible version.

If you need assistance or would like to learn more, please contact your Edwards Sales representative or call Edwards Technical Support: 1-800-822-9837

For more than 50 years, Edwards Lifesciences has been helping you make proactive clinical decisions and advance the care of surgical and acutely ill patients across the continuum of care. We are committed to evolving our solutions to expand the benefits of advanced hemodynamic monitoring to more patients.



1. EV1000 Monitor
2. EV1000 Pump-unit
3. EV1000 Patient monitor adapter cable
4. Heart reference sensor

**CAUTION:** Federal (United States) law restricts this device to sale by or on the order of a physician. See instructions for use for full prescribing information, including indications, contraindications, warnings, precautions, and adverse events.

Edwards, Edwards Lifesciences, the stylized E logo, ClearSight, EV1000, and HemoSphere are trademarks of Edwards Lifesciences Corporation or its affiliates. All other trademarks are the property of their respective owners.

© 2021 Edwards Lifesciences Corporation. All rights reserved. PP-US-6337 v1.0

Edwards Lifesciences • One Edwards Way, Irvine CA 92614 USA • edwards.com

