



Two implant options to treat your MR patients

Giuseppe Tarantini, MD, PhD, FESC

Director Interventional Cardiology

University of Padua Medical School, Padua (Italy)



Potential conflicts of interest

Speaker's name: Giuseppe Tarantini, MD, PhD

☐ I have the following potential conflicts of interest to report:

Lecture Fees for Edwards LifeSciences


Expert opinion disclaimer

- Expert opinions, advice and all other information expressed represent contributors' views and not necessarily those of Edwards Lifesciences.

DO WE REALLY NEED 4 SIZES? EXPAND G4 Registry¹


N° OF IMPLANTED CLIPS/PTS²

96% ≤ 2 clips per pts

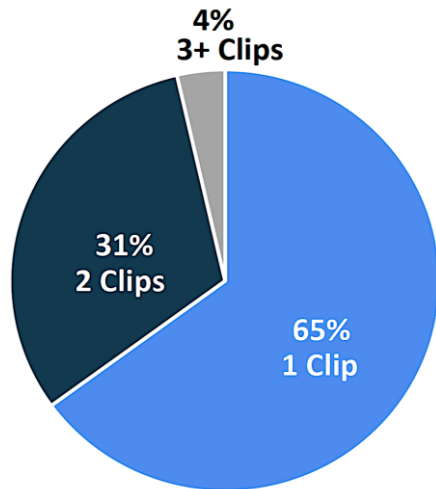

14 CLIP COMBINATIONS²

66% used NTW or XTW only

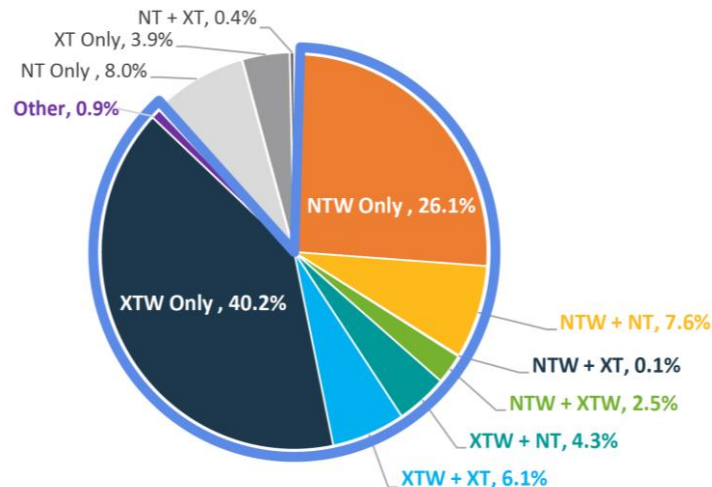

88% used NTW and/or XTW³


→ Mostly, only two implant options are used

¹Multicenter registry, with 60 sites, 1164 patients enrolled, 42% having degenerative mitral regurgitation. ²1141 subjects with at least 1 clip implanted

³Von Bardeleben R.S., Contemporary Clinical and Echocardiographic Outcomes of 1000+ Patients Treated with MitraClip™ G4: Results from the EXPAND G4 Post Approval Study. TCT 2022
Based on: Tarantini G., Two implant options to treat all your MR patients. Edwards TNT “Treating diverse mitral regurgitation anatomies: the power of versatility”, EuroPCR 2023, May 16th.

DO WE REALLY NEED 4 SIZES? EXPAND G4 Registry¹

PRECONCEPTs ABOUT THE NEED FOR MORE OPTIONS

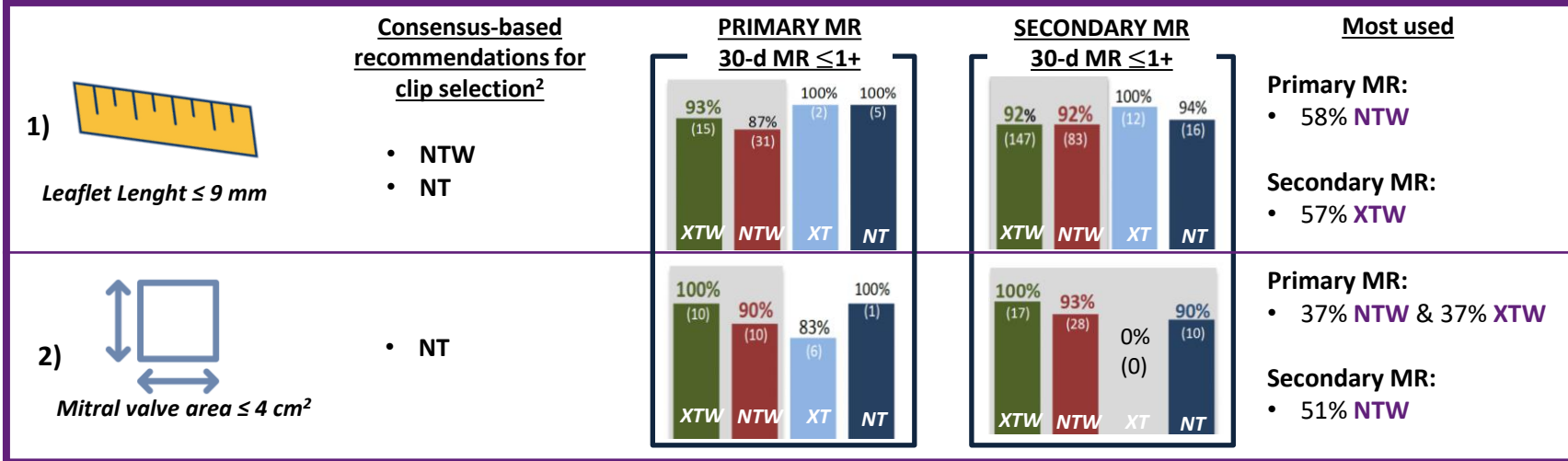

XTW and NTW are preferred as 1st clip option², despite the recommendations for clip selection
No significant differences in MR reduction, despite the use of different clips


¹Multicenter registry, with 60 sites, 1164 patients enrolled, 42% having degenerative mitral regurgitation. MR: mitral regurgitation

²Maisano F. et al., Clip Selection Strategy with 4th Generation MitraClip™: Evidence-Based Recommendations from the Global EXPAND G4 Study. PCR London Valves 2022

Based on: Tarantini G., Two implant options to treat all your MR patients. Edwards TNT “Treating diverse mitral regurgitation anatomies: the power of versatility”, EuroPCR 2023, May 16th.

Edwards PASCAL Transcatheter Valve Repair System*

PASCAL Implant

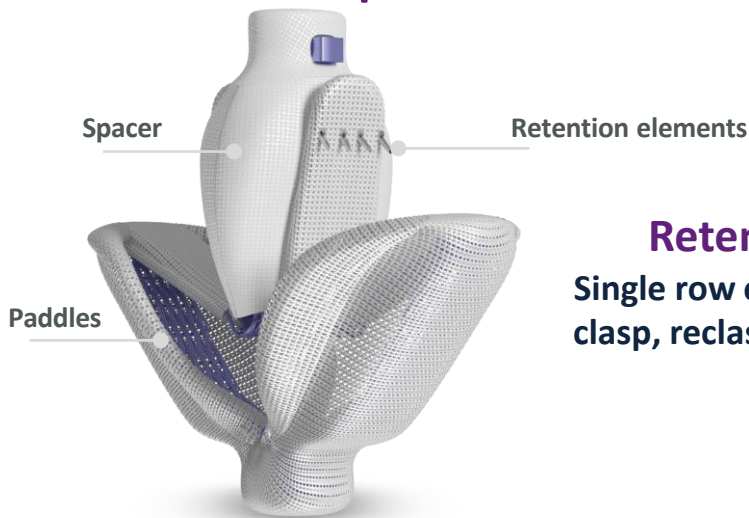


Retention elements

Single row of retention elements to clasp, reclip, and preserve leaflets

**Wide contoured paddles with
a larger central spacer**

**Close the implant to conform to native anatomy
and flex during the cardiac cycle**

PASCAL Ace Implant

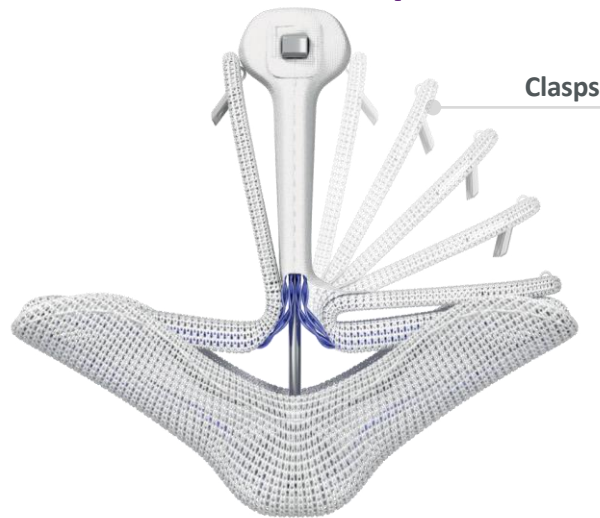

**Independent grasping &
atraumatic clasp and closure**

**Allow for staged leaflet capture and adjustment
and help you preserve leaflet integrity**

* Performance, design and simulation data on file. Note: Images are not actual size


OVERARCHING PRINCIPLES FOR DEVICE SELECTION

- 1) Ensuring adequate MR reduction for both FMR and DMR
- 2) Provide preservation of MVA and stability of the implant with adequate leaflet capture

Finite Element Analysis (FEA) in a mitral valve simulates leaflet response to an implant

Implant without spacer


Open orifice area = 156.3 mm^2 *

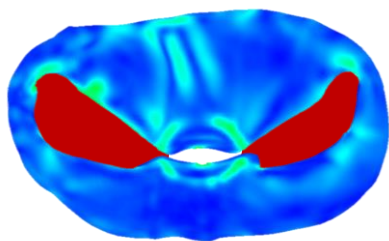

- Lower stress concentration
- Higher stress concentration
- Diastolic open orifice area

PASCAL implant

Open orifice area = 188.1 mm^2 *

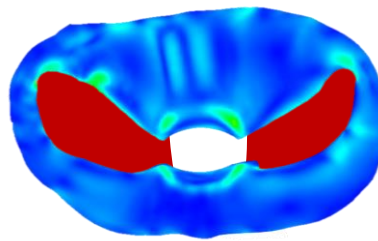

- ↓ Reduced stress on leaflets *
- ↑ 20% increase in open orifice area during diastole *

* Simulation data on file; MR: mitral regurgitation; FMR: functional MR; aFMR: atrial FMR; DMR: degenerative MR; MVA: mitral valve area

Expert opinions, advice and all other information expressed represent contributors' views and not necessarily those of Edwards Lifesciences.

TWO IMPLANT OPTIONS FOR YOUR MR PATIENTS

Latest experts' recommendations¹ EuroIntervention 2023

Anatomical features		NT	XT	NTW	XTW	PASCAL	PASCAL ACE
Length of the mobile leaflet in the grasping zone	<9 mm	✓		✓		✓	✓
	>9 mm		✓		✓	✓	✓
Broad gap size				✓	✓	✓	
Small MVA (<4.5 cm ²)		✓		✓			✓
Thin leaflet structure		✓		✓		✓	✓
Commissural jet		✓		✓			✓
Barlow's disease			✓		✓	✓	✓
MVA: mitral valve area; PMR: primary mitral regurgitation; SMR: secondary mitral regurgitation							

PASCAL Ace implant seems to be the workhorse

MR: mitral regurgitation

¹Hausleiter J. et al. Mitral valve transcatheter edge-to-edge repair. EuroIntervention. 2023 Jan 23;18(12):957-976

Expert opinions, advice and all other information expressed represent contributors' views and not necessarily those of Edwards Lifesciences.

Thank You!

Thank You!

Giuseppe.tarantini.1@gmail.com



@interventionalcardiologyteampadova



@giuseppetarantinimdphd



@G_Tarantini01

Thank You!

Expert opinions, advice and all other information expressed represent contributors' views and not necessarily those of Edwards Lifesciences.

Medical device for professional use. For a listing of indications, contraindications, precautions, warnings, and potential adverse events, please refer to the Instructions for Use (consult eifu.edwards.com where applicable).

Edwards, Edwards Lifesciences, Edwards PASCAL, PASCAL and PASCAL Ace are trademarks or service marks of Edwards Lifesciences Corporation. All other trademarks are the property of their respective owners.

© 2023 Edwards Lifesciences Corporation. All rights reserved. PP--EU-6283 v1.0

Edwards Lifesciences • Route de l'Etraz 70, 1260 Nyon, Switzerland • edwards.com

PCR

PCRONline.com