



Made in USA

FLEXcon® Face Shields

Single-use Face Shields For General Healthcare Use



PRODUCT INFORMATION

Product FLEXcon Face Shields

Indications for Use

Single-use face shield intended for general healthcare use; not intended to provide liquid

Single-Use Face Shields

barrier protection.

Part Number FLX068549

Product Dimensions 18" x 10"

Thickness 5 mil (0.005 inch)

Color Clear

Material Polyester Film

Clarity 99.7% Haze < 1%

Manufactured Spencer, MA

FEATURES & BENEFITS

Disposable single use product **Full Face Shield**

that covers and protects the

face.

Full Visibility Optically clear face shield

> plastic barrier that provides unrestricted visibility and wraps

around the face.

Premium Materials Made of high quality clear

polyester film.

Lightweight & Transparent, lightweight, can

Comfortable be worn over a face mask or

other facial covering.

Flexible Comes with holes and can be

installed on any pair of safety or

prescription glasses.

Easy Installation Easy to install and remove.

WARNINGS

Contents

- Discard face shield immediately if torn, cracked, or damaged.
- Face shields are non-sterile and should not be used in any settings where sterility is required.

DISCLAIMER

These single-use disposable face shields are manufactured by FLEXcon and intended for general healthcare use; not intended to provide liquid barrier protection. They are not tested for medical use and may not prevent the spread of infectious disease.

- The product has not been FDA cleared or approved.
- The product has been authorized by FDA under an EUA for use by healthcare providers as personal protective equipment.
- This product is only authorized for the duration of the declaration that circumstances justifying the authorization of emergency use under Section 564(b)(1) of the Act, 21 USC 360bbb-3(b)(1) unless the authorization is terminated or revoked sooner.

BODY CONTACTING MATERIALS

PFT Film

CAUTIONS/CONTRAINDICATIONS

NOT for use in:

- Any surgical settings or where significant exposure to liquid, bodily or other hazardous fluids, may be expected;
- Clinical settings where the infection risk level through inhalation exposure is high; or the presence of a high intensity heat source or flammable gas.

Face shields are not intended to provide antimicrobial, antiviral, or infection protection or prevention.




FOR ORDERS

Please visit our webpage by scanning the QR code or visit the following URL www.flexcon.com/products/face-shields







FLEXcon® Face Shields | Single-use Face Shields For General Healthcare Use

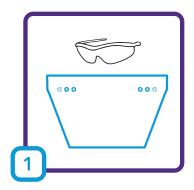

Unpack Face Shield from box and locate a pair of safety glasses to attach to the Face Shield.

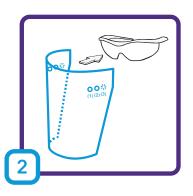

Bend the Face Shield sheet as indicated.

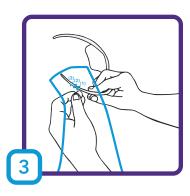


Start with putting one temple of the safety glasses through the 1st hole (1) weaving it through to the 2nd hole (2).

Make sure the material for the Face Shield is on the inside.



Repeat Step 3 on the opposite side.

If using regular eyeglasses, the 3rd hole (3) can be used. You may need to trim part of the shield around the ear for optimum comfort and fit.

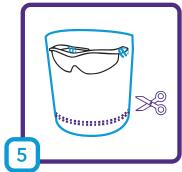

Cut the Face Shield sheet to your desired length.

The Face Shield is ready



Made in USA



Patent Pending

DISCLAIMER

These single-use disposable face shields are manufactured by FLEXcon and intended for general healthcare use; not intended to provide liquid barrier protection. They are not tested for medical use and may not prevent the spread of infectious disease.



FOR ORDERS

Please visit our webpage by scanning the QR code or visit the following URL www.flexcon.com/products/face-shields