




HemoSphere Advanced Monitoring Platform Setup guide

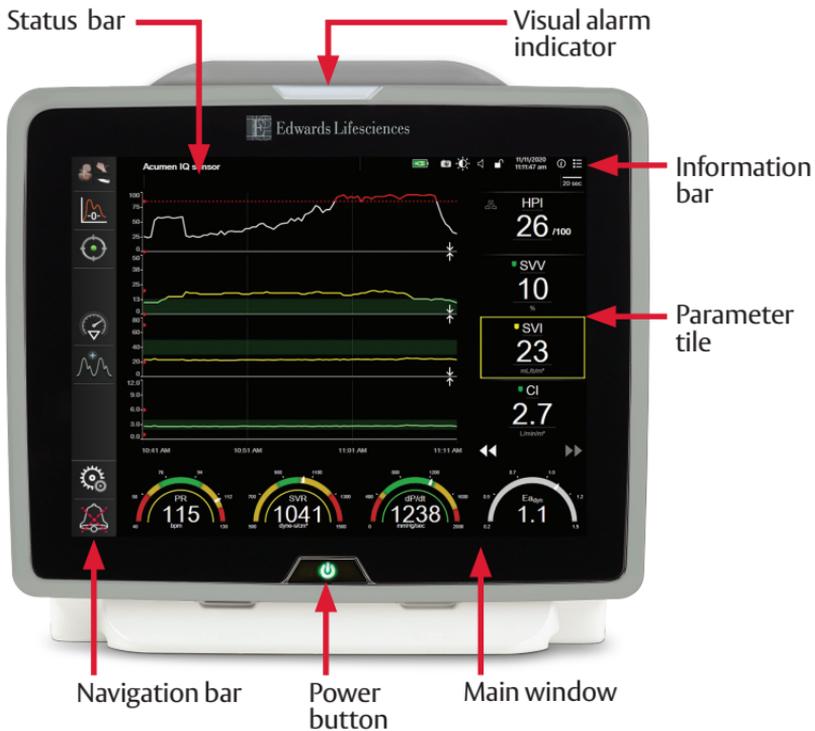

HemoSphere advanced monitor screen features

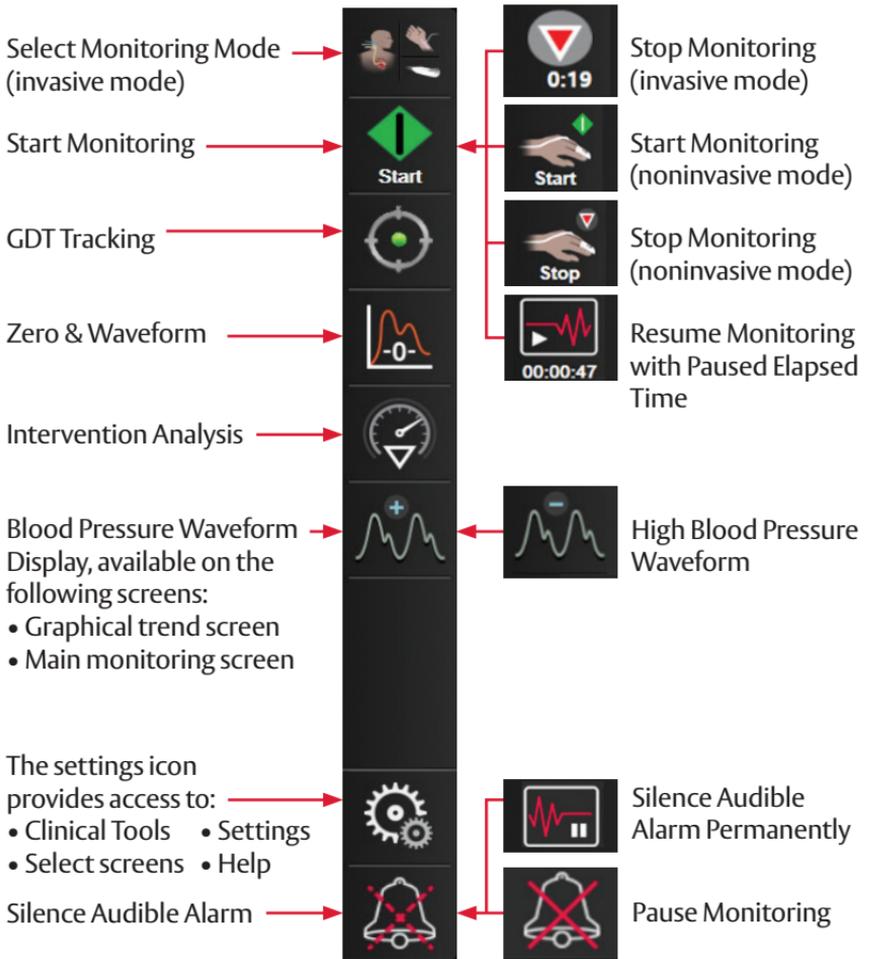

Information bar (located at the top of the screen)

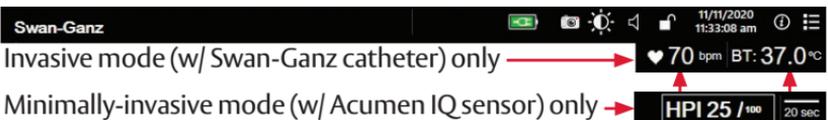

Clinical actions

Changing parameters

1. Touch the displayed parameter label located inside the parameter tile to change it to a different parameter.
2. The tile configuration menu will show the selected parameter highlighted and other parameters currently being displayed outlined. Available parameters appear on the screen without highlight.
3. Select desired parameter to display.







Changing targets and alarms

1. Touch the parameter value on a parameter tile.
2. Adjust target values using arrows or select value and type in parameter target.
3. Select **Target** to disable or enable colors in target ranges.
4. To silence parameter alarms, select **Audible Alarm** button.



Monitoring Pause

1. Touch the silence audible alarms icon continuously for five seconds to show additional alarm silencing options.
2. Touch this icon to pause monitoring. A monitoring pause confirmation banner will appear to confirm suspension of monitoring operations. Exception: Tissue oximetry monitoring and associated alarms will remain active during a monitoring pause.
3. After monitoring pause confirmation, a resume monitoring icon and elapsed time will appear on the navigation bar. A "Monitoring Pause" banner will be displayed. To return to monitoring, touch the resume monitoring icon.

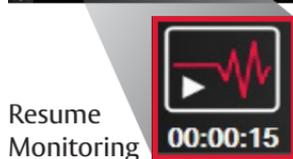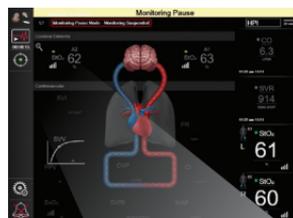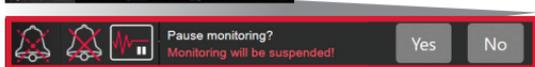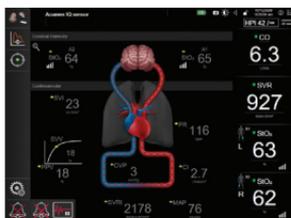

Zeroing

Open the sensor stopcock valve to measure atmospheric conditions.

1. Press and hold the physical zero button directly on the pressure cable.
2. When zeroing is complete, a tone sounds, and the message "Zeroed" appears with the time and date.

Or

Touch the zero button located on the screen.



CAUTION: Federal (United States) law restricts this device to sale by or on the order of a physician. See instructions for use for full prescribing information, including indications, contraindications, warnings, precautions and adverse events.

Edwards, Edwards Lifesciences, the stylized E logo, Acumen, Acumen IQ, HemoSphere, Swan, and Swan-Ganz are trademarks of Edwards Lifesciences Corporation or its affiliates. All other trademarks are the property of their respective owners.

© 2021 Edwards Lifesciences Corporation. All rights reserved. PP--US-6139 v1.0

Edwards Lifesciences • One Edwards Way, Irvine CA 92614 USA • edwards.com



Edwards